

PROCEDURE 2026 – MAGNETIC TATTOO REMOVAL -UPDATED

Before the Treatment

1. Request the **photos** from your client and case history (if any) to study and understand the case.
2. Complete the **consent form** with detailed information about the tattoo's history and any previous removal methods, if applicable.

The day of the treatment

3. The day of the treatment:

Consultation - Advice and explanations aftercare

- Explain the entire procedure to your client, from start to finish, in detail. Allow them to ask questions, show them the device, the solution, some healing progress pics so they know exactly what to expect.
- Explain the aftercare and Treatment process in detail. Remind them that the epidermis requires 5 days to heal, and that the full healing of the skin takes 2-3 weeks.
- Check for titanium ink in the skin and for the presence of scars from previous removal methods.
- In the case of presence of titanium ink/scarring/very little hair, client must sign a form of acknowledgement of scarring/titanium ink/very little hair. Include this in your consent form.
- Give the client a post care brochure that can be taken home and referred to during the healing process.

Treatment Process

4. Treatment:

- Clean Clients brows thoroughly with a Saline water-soaked cotton pad
- Eyes' clients can be protected with cotton pads soaked with rose water, for better comfort.
- Work thoroughly on the entire brow or lip area to ensure that the skin is properly prepped.

Client's Before Pics and videos:

- Take photos always after cleaning the area.
- Ensure the use of good lighting and proper photography technique to obtain clear and accurate pics and short videos.
- Always take pics at each session. Use the same lighting and angles as used in the initial session to ensure accurate measurement of progress.

5. Prepping the skin with the machine

- Insert sterile needle 132 magnetic into device after removing the tiny metal tip protector first.
- Switch on the device, do not adjust the speed as it is already set for tattoo removal use.
- Adjust needle to 1.5-2mm above the edge of the needle cartridge.
- Light exfoliation: Gently slide the device across the entire brow to remove dead skin cells.
- Use back and forth, up, and down movements throughout the brow for about 15 seconds per brow.

IMPORTANT: Use dry needle, no solution or E-Dermis

6. In case the skin is mature or dry, Apply E-Dermis aftercare ointment to both brows and cover with plastic film for about 2 minutes. Remove ointment from both brows with clean tissue to avoid working on oily skin.

7. Apply a thin layer of Ink extractor solution across the entire brow, both sides. Use the device to pick up product and slide gently across each brow, staying within the brow borders,

WHILST DEVICE IS OFF.

Leave to dry until it becomes a white powder, then use a wet cotton pad to gently remove powder from one brow. This is the brow you will be working on first.

8. Protect/cover the eyes using cotton pads dipped in organic rose water to avoid product splashing into the eyes.

9. Secure the brow and forehead area firmly in place with your hand and fingers to ensure that the vibration of the machine is lessened, causing less discomfort, and greater accuracy of movement.

10. Switch the device on and start working from the MIDDLE of the brow. Use brisk, but gentle, backs and forth, and up and down movements throughout the MIDDLE of the brow.

When working on the tail and at the beginning of the brow, use **mainly up and down movements**, with few backs and forth movements.

- Always be careful to work more up and down than back and forth at the beginning and tail of the brows. Also, where skin is damaged.
- Dip your needle in product regularly during the procedure!!!

In case you did not protect the eyes with cotton pads soaked with rose water, rinse the eye with normal saline serum.

- Skin that has been scarred or has undergone invasive procedures such as tattooing, microblading, or removal treatments using acid, saline, or high-alkaline solutions may have micro-scarring in the deep dermis. This can lead to pinprick bleeding during the procedure.

To address this, gently pinch the skin between your fingers to help expel any underlying blood clots from the damaged dermis. Wipe the area with a wet cotton pad.

Regularly applying firm pinching throughout the procedure will aid in expelling as much of the blood clots as possible, while also helping to reduce redness in the skin.

11. Use a dry cotton roll to gently clean the brow. Massage with E-Dermis and do second round on same brow.

- Do a minimum of 5 to 6 rounds, using your discretion based on the condition/quality of the skin.
- Clean the brow with a Saline water-soaked cotton pad massage with E-Dermis and place another cotton pad soaked with saline water over the brow. repeat entire procedure on the other brow. While working on the second brow, periodically massage the first brow with E-Dermis.

12. After completing the tattoo removal process on both brows, apply a thin layer of E-Dermis Boost Tattoo Removal Mask to the treated area.

This mask provides multiple benefits:

- Hydrates the skin and helps to close the pores.
- Soothes and cools the skin that was heated by the treatment.
- Assists in extracting Nanoparticles of pigment from the upper skin layers.
- Helps to release the micro blood clots that were left behind and not removed through manual squeezing.

MAGNETIC
TATTOO REMOVAL
BY LINDA PARADIS

Its advanced formula works gently to extract residual pigment that has surfaced to the upper dermis, enhancing the effects of the E-Dermis Ink Extractor solution and the non-invasive magnetic needles used during the procedure.



13. Let the mask dry 2-3 min max then with a Saline water-soaked cotton pad or facial sponge, clean the eyebrows from the mask.

14. For the lips, do not apply the Boost removal mask and put little E-dermis aftercare.

15. Do not apply any aftercare, client go home with dry eyebrows. to avoid dust stick to the aftercare.

16. Give the client clean tissues to wipe lymph fluid off the brows on the way home.

17. Once the client is at home, they must wash their face with a gentle face wash and apply E-Dermis aftercare.

- At home, in the case of previous scarring from other methods or invasive PMU or Microblading, the client must use any antibacterial ointment after washing her face for few days.
- Maintain close follow-up with the client in the days following their first session to provide reassurance and remind them of the aftercare protocol and healing process.
- Reassure them that scabbing for 5-7 days is a normal part of the healing process and not a cause for concern.
- No makeup on brows for 2 weeks, thereafter only powder makeup, not pencils.
- No touching or picking at the scabs and always keep the brow area clean and sterile.

18. Clients must apply the E-dermis cream 4-5 times per day.

If there was scarring from previous invasive treatments such as Microblading, saline removal, TCA, or laser, they must apply Fucidin or Neosporin antibacterial ointment for



few days, then Aquaphor or Mebo cream both after showering or washing face for few days, one week later they start E-dermis aftercare.

19. Ask for daily progress pics to reassure the client and remind them that there is nothing to worry about.

20. Explain in the case of the brows becoming pink after healing, that it is due to the orange/pink/red/brown pigments from previous treatments/ from previous scarring/ blood clots from previous damage that have come to the surface and will need to be squeezed out on future sessions.

21. Next session can be booked 3 to 4 weeks later. It is better to immediately give the next appointment to the client. Remind again the post care instructions, and to not worry at all about any redness during the healing process,

Linda Paradis